

“The Bobtail can be used by men or women.”

Men can use a public restroom urinal without having to pull the ostomy bag out.

Women can use the Bobtail to control flow directly into the toilet. All can use the Bobtail for discreet emptying into a bottle as needed.



GV Solutions, LLC



Camp Hill, PA 17011,



717-319-5424



robertgreen58@gmail.com

For detailed explanation and demonstration of the Bobtail, go to:

<https://youtu.be/q0fO7EXePM8>



Bob Green
Presents

“The Bobtail”

A tube extension with a valve for discreet and easy elimination of urine from an ostomy bag

Design Patent D978,345.

A new and refined way to empty an ostomy bag.



The bags empty the same way for men or women, draining through an opening at the bottom of the bag. This requires the bottom opening be held while aiming for the disposal spot.



The Bobtail changes that. By adding a short connector with a valve, the Bobtail can be pulled through a zipper opening, allowing the valve to be opened and the urine drained.

The Bobtail can be customized to accommodate any distance from the ostomy bag opening to the desired length. The Bobtail can come with a corrugated tubing for easy bending, or have a more rigid tube as shown below



The Bobtail easily connects to any ostomy bag having either a valve or a plug.

Up to now, all ostomy bags have a cumbersome way to be emptied.

The bag must be pulled out from one's clothing to be emptied, while standing or sitting.

This requires that the bag be held with one hand as a plug is removed or the valve opened to allow the urine to drain.

The Bobtail can be worn pulled through the underpants for easy access or worn entirely within the undershorts.



Either way the Bobtail can be pulled through the zipper opening with the valve in easy reach.



The Bobtail consists of a connector to the ostomy bag with a valve and a cap.

The valve is easily opened and closed.